

QUENTRY ON THE MAC

Software Versions: Qentry 2.x, TraumaCad Web

REQUIREMENTS

- Mac OS: 10 or higher

TO RUN QUENTRY.COM:

- Safari version: 7 or higher
- Firefox version: 20 or higher

TO ACCESS QUENTRY GATEWAY:

- Safari version: 7 or higher
- Firefox version: 20 or higher




UPLOADING AND VIEWING IMAGES ON QUENTRY


- To upload and view images on **Qentry.com**, ensure Microsoft Silverlight is installed using: <http://www.microsoft.com/getsilverlight/Get-Started/Install/Default.aspx>

ENABLING SILVERLIGHT


Configure Silverlight for Qentry as follows:

1. Open the **Qentry.com** website in Safari.
2. Select **Preferences** under **Safari**.
3. Select the **Security** tab.
4. Make sure the **Allow Plug-ins** checkbox is activated.
5. Click **Manage Website Settings...**
6. Select **Silverlight** in the plug-in list on the left.
7. Click on the pull-down for the Silverlight enabled website and choose **Run in Unsafe Mode** ①. This allows Qentry content to be used correctly and to enable certain system functionalities inside Silverlight.

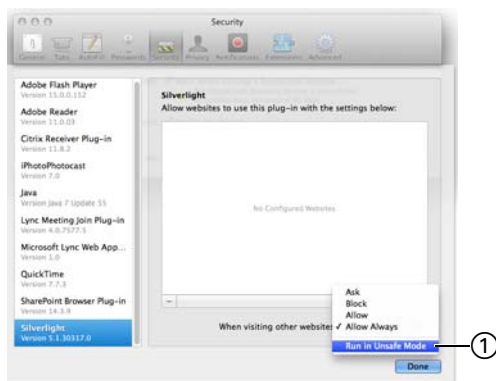

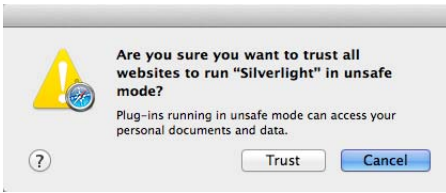


8. In the pop-up, click **Trust**.
9. Click **Done**.
10. Close the **Preferences** panel.
11. Log in to **Qentry.com** using your Brainlab ID to ensure correct functionality.

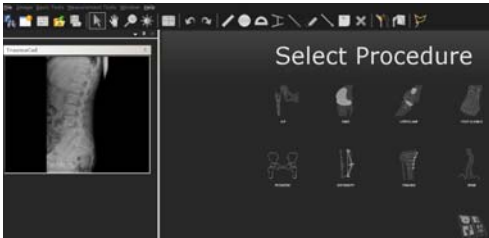

TEMPLATING WITH TRAUMACAD WEB

When accessing a patient folder, click on the **TraumaCad** icon. **TraumaCad Web** opens.

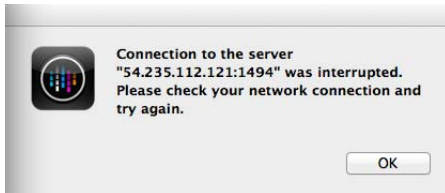

IT SETUP

If you receive the failure message:
"Connection to the server ... was interrupted"

Contact your IT department and request that the following items be approved by your site's Firewall to allow proper communication to/from Qentry when using **TraumaCad Web**:

- FQDN: C01-US1.QENTRY.COM
- IP Address: 54.235.112.121
- Port used: SSL - 443, Citrix 1494, 2598

MAC WITH PARALLELS (WINDOWS PARTITION)

Qentry has been tested in this mode and it works in the same way as on Microsoft Windows.

For further assistance please contact support@qentry.com.

MANUFACTURER INFORMATION:

Brainlab AG
Kapellenstr. 12, 85622 Feldkirchen, Germany

Europe, Africa, Asia, Australia: +49 89 99 15 68 44
USA and Canada: +1 800 597 5911
Japan: +81 3 3769 6900
Latin America: +55 11 33 55 33 70
France: +33-800-67-60-30

E-mail: support@qentry.com

COPYRIGHT:

This guide contains proprietary information protected by copyright. No part of this guide may be reproduced or translated without the express written permission of Brainlab.

Document Revision: 1.0

Article Number: 60916-36EN

LIABILITY:

This guide is subject to change without notice and does not represent a commitment on the part of Brainlab.

For further information, please refer to the "Limitations of Liability" section in the Brainlab Standard Terms and Conditions of Sale.

